

Medical Face Mask

Suitable for medical personnel

- - Category I medical device
- - Tested according to EN 14683
- - Available as Type I, Type II and Type IIR
- - Suitable for medical professionals and for wearing in public
- - High wearing comfort
- - Extremely light



Medical face masks reduce the spread of bacteria and viruses when speaking, sneezing, coughing and breathing out. The mask consists of three layers and is comfortable to wear due to its flexibility and odorlessness. In contrast to face filtering respirators, the breathing resistance of the mouth-nose protection is significantly lower, which also contributes to wearing comfort.



Do not reuse



Attention, see instructions for use



Don't use when packaging damaged



See information supplied by the manufacturer



Temperature range of storage conditions



Maximum relative humidity of storage conditions

STANDARDS AND APPROVAL

This medical face mask has been tested in an accredited test laboratory of the company SGS according to the standards EN 14683 and EN 10993 and classified as a category I medical product.

APPLICATION

1. clean your hands thoroughly
2. do not contaminate the inside by touching it
3. pull the two rubber bands over the ears
4. place the mask correctly over mouth, nose and cheeks, so that the edges are as tight as possible
5. wash your hands thoroughly after taking off
6. dispose of the mask in a sealed bag

INFORMATION

- for single use only - cleaning not possible
- immediately remove a soaked mask and replace it if necessary

Performance Requirement EN 14683

TEST	Type I	Type II	Type IIR
Bacterial Filtefficiency (BFE), (%)	≥ 95	≥ 98	≥ 98
Differential pressure (Pa/cm ²)	< 40	< 40	< 60
Synthetic Blood (kPA)	-	-	≥ 16,0
Microbial Cleanliness (cfu/g)	≤ 30	≤ 30	≤ 30

Technical Datasheet